## Activinsights Band App Instructions

### ENGLISH

The Activinsights App connects via Bluetooth to your mobile device and regularly uploads your data to our server. The app is compatible with Microsoft, Android and Apple devices. Please see steps below on how to download the app for your device type.

#### PC Application (Microsoft)

1. Search for Activinsights Band Uploader in the Microsoft Store and install it.
2. Once complete, launch the application.
3. Press and hold the silver button on the Activinsights Band for five seconds until the app recognises the band.

#### Android (Google Play)

1. Ensure that your Bluetooth is turned on.
2. Search for Activinsights Band Uploader in the Google Play Store and install the app.
3. Once installed, open the app. *Note: The app may ask for permission to access your location and notifications which are required for the connection to the Activinsights Band. Please allow this for full functionality.*
4. To connect your Band, press and hold the silver button on the Band for five seconds and then release.
5. The Band will upload data automatically providing the app is running. You will not need to manually reconnect the Band each time you use the app.

#### Apple iOS (App Store)

1. Ensure that your Bluetooth is turned on.
2. Search for Activinsights Band Uploader in the App Store and install the app.
3. Once installed, open the app. *Note: The app may ask for permission to access your Bluetooth and notifications which are required for the connection to the Activinsights Band. Please allow this for full functionality.*
4. To connect your Band, press and hold the silver button on the Band for five seconds and then release.
5. Data will then begin to upload. Leave the app open on your device for uploads to continue. If the app is closed, it will need to be reopened for the Band to upload data.
6. The Band will continue to upload data automatically providing the app is open and running. You will not need to manually reconnect the Band each time you use the app.

## FAQs & Troubleshooting

### **Why isn't the app connecting to my Band?**

- Make sure app is open and running
- Ensure the main screen isn't reporting any issues
- Check notifications for any error messages
- Ensure Bluetooth is connected
- Ensure you have accepted all permissions
- If the band still won't connect, press and hold the silver button for five seconds to start a manual upload

### **Why hasn't my Band uploaded any data?**

- Make sure the app is open and running
- Check mobile data is turned on
- Ensure Bluetooth is connected
- Check the Uploads tab for error messages in red. If an error has been displayed for more than 24 hours, go to the About tab and press Upload Support File.

### **How do I upload data?**

- The app needs to be open and running in the background in order to upload data. If the app is force closed it will not upload any data.
- Once you have initially connected your band to the app, it should automatically upload data providing the app is running.
- If your data does not automatically upload, you can do this manually by pressing the button on the band for five seconds and releasing.

### **Will the app drain my phone battery?**

- If you do not usually have Bluetooth running on your device you may notice that your battery drains slightly quicker than normal, however this will be very minimal.

### **Troubleshooting Android Issues**

- If you experience problems with your Android device or your phone switches off, ensure the app is still running.
- The Uploads tab will show any error messages that may have occurred.

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